Savanah Bath Step

	New Code	Old Code
Retail	09 107 7510	AA1824Y
Non-Retail	09 107 7502	AA1824W
Pack of 12	09 116 2379	
Replacement Pad	09 107 7478	AA182401

Maximum user weight  

Instructions for Use

1. Remove the bath step and pad from the packaging.
2. Fit the plugs (attached to the underside of the step) into the holes in the centre of the step.
3. Attach the self adhesive, slip resistant pad:
 - a) Ensure the step is dry and free from dust and grease.
 - b) Peel off the backing sheet along one long edge of the pad and align it with the recess in the top of the step.
 - c) Once fitted squarely in the recess, pull off the rest of the backing sheet and smooth down the pad.

The Savanah Bath Step is designed to stack (maximum 6 steps), clip together or a combination of the two.

Adding steps

1. Follow the Instructions for Use for all steps that will be at the top of any stacks.
2. For steps within the stacks, do not fit the plugs or self adhesive pad.
3. Stack the steps to the height required (maximum 6 steps) ensuring that each one is properly located on the one below. Place the prepared top steps on last. Clip single steps and stacks together as required.

Care and Cleaning

Clean with warm water and mild detergent or household disinfectant.

Cautions

- Ensure all the top steps are assembled with the plugs and pad fitted correctly before use
- Do not stack steps more than 6 high
- Maximum user weight 190kg / 30st
- Do not use on a wet floor
- Replace the foam pad when it becomes worn or ingrained with dirt as this may reduce the effectiveness of the slip-resistant surface.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

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