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Assembly and operating instructions /
Montage et d'utilisation /
Montage- en gebruiksaanwijzing

Technische Daten / Technical details / Caractéristiques techniques / Technische gegevens:

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Shanghai Maxcare Medical Device Co., Ltd.

No. 93, Dianxing Rd., Dianshanhu Town, Kunshan City, Jiangsu Province, P.R. China, 215345

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Assembly:

Attach the middle piece to the side part and firmly fasten using a screw, washer and screw nut. The pins on the middle piece should point forwards. Then the seating surface is to be latched onto the frame by pressing down.

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Height adjustment:

The shower seat height is adjusted on the legs. To do so, the retaining clip must be removed. Then the telescopic tube can be retracted to the desired length. Finally, the retaining clip is fully latched on once again.

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EN

Intended use:

The shower chair is a medical device and is designed as a sitting and standing aid for people with limited stability/function of the lower extremities. The hygiene cutout can facilitate or enable body care in the shower. However, the shower chair without a backrest may only be used by people who can independently control their balance while sitting, all products should only be used with the support of a caregiver.

Indication:

Limited ability to stand, limited stability, general weakness

Contraindication:

Dizziness, impaired balance function, lack of trunk control, states of confusion, insufficient physical or mental abilities for using it

Item description / Versions:

The shower chair is used to carry out personal hygiene while sitting in the shower in a comfortable and safe way. It is made of aluminium and plastic and can be adjusted to four different sitting heights. The seat is equipped with a non-slip surface, the hygiene cutout facilitates body care and intimate hygiene. There are different models (see technical data), the selection of a suitable chair should be made together with specialist personnel. Testing, assembly and instruction in use may only be carried out by authorised, trained specialist personnel. The item should only be used with the help of an accompanying person/caregiver. The model is designed for domestic use, the model must only come into contact with intact, uninjured skin and may not be used for more than one patient. Any other or additional use other than the one mentioned above must not be used. SUNDO GmbH or the manufacturer are not liable for any resulting damage; the user bears the sole risk. The manufacturer or competent authority must be notified immediately of all serious incidents relating to this product.

Before use / Maintenance:

First read these instructions for use completely and keep them with the packaging until the end of the product's life. Please check that the product is undamaged before the first use and each time it is used. Also check the safe position of the shower chair before each use, regularly check the screw connections and the stability of the seat. If there are signs of damage or wear, the shower chair must not be used (any more). Further maintenance is not required if the safety instructions are observed.

How to use:

Before use, the carer must ensure the stability of the seat. If the seat tilts, adjust it in such a way that it is stable. The carer must always provide support for sitting down and standing up. Always sit upright and in the middle of the shower seat, if not, there is a danger of tipping over or slipping!

Cleaning and disinfection:

- The shower chair can be cleaned with standard household cleaning agents, however, ensure that the cleaner does not contain oils, fats or abrasives.
- The apparatus can be disinfected in the wipe disinfection process using commercially-available chlorine-free disinfectant.

Important: When using disinfectants, always follow the manufacturer's instructions for use.

⚠ Safety instructions:

- Product and packaging are not toys, please do not leave children unattended with it!
- Use only by persons whose physical and mental abilities are sufficient for this purpose, under the supervision of a carer.
- Body weight of user: maximum of 120 kg!
- There is a danger of crushing if hands or extremities are placed in the gap between the frame and seating surface!
- There is also a danger of crushing during assembly and when collapsing the apparatus.
- The shower seat should not be used by persons who are incapable of controlling their balance while sitting.
- The person providing assistance must ensure the stability of the apparatus and check that the side frame is correctly placed in the side frame mount before using!
- A person must be present to provide assistance when sitting down on the shower chair and standing up!
- If parts of the shower chair are bent or warped, it can no longer be used!
- All 4 feet must be adjusted to the same height and the shower chair must sit firmly on all 4 feet!
- Especially with the foldable model, you must make sure that the chair is completely folded out and fixed.
- Always use and strain **both armrests** evenly, otherwise there is a risk of tipping and falling!

Warranty: In accordance with statutory warranty rights. Excluded from the warranty: Damage caused by improper use, normal wear and tear, and defects which do not significantly affect the value or usability of the device. Technical modifications and attachments, as well as improperly performed maintenance void the warranty and product liability in general. We reserve the right to make modifications aimed at improving technical features.