# **HENROCUP®**

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**HENROCUP® INSTITUTION UND DECKEL**

**JANUAR 2023**

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**JANUARY 2023**

**Table of content**

<b>Important information .....</b>	<b>3</b>
<b>Intended use .....</b>	<b>3</b>
<b>Storage .....</b>	<b>3</b>
<b>Instruction for use .....</b>	<b>4</b>
<b>Precautions.....</b>	<b>4</b>
<b>Daily operation.....</b>	<b>4</b>
<b>Warnings.....</b>	<b>5</b>
<b>Environmentally friendly disposal .....</b>	<b>5</b>
<b>Technical datasheet .....</b>	<b>6</b>
<b>Technical datasheet .....</b>	<b>7</b>

## **Important information**

Before using HENROCUP® institution read these instructions closely.

## **Intended use**

HENROCUP® Institution is designed and intended for beverage and food only.

HENROCUP® Institution has been developed by Danish occupational therapists to ensure a high-quality cup for people that experience loss of strength and coordination. HENROCUP® Institution is BPA Free and can safely be used by children under the age of 3 years.

HENROCUP® Institution is design with 2 wide handles to give a firm grip, the shape of the cup and handles ensures that you can drink without bending the neck. The bottom of the cup is wide to help it stand firm on the table and reduces spillage.

HENROCUP® can with advantage be used in hospitals, nursing homes, residences and special institutions.

## **Storage**

Store dry, at room temperature and out of direct sunlight.

## **Instruction for use**

Content: 1 HENROCUP® can come with or without lid.

1. Unpack the cup and lid, dispose of the packing properly.
2. Before use wash the cup and lid.
3. The HENROCUP® are now ready for use.

## **Precautions**

After unpacking the cup always wash the HENROCUP® Institution before use.

The cup intended use are for drinks.

Warm drinks can cause miscoloring and diminish the lifespan of the cup.

Do not heat the cup over a candle light or fire.

Be careful with hot fluid.

If used with lid, do make sure before use to check that the lid is put on correctly and is not loose.

## **Daily operation**

HENROCUP® Institution can be washed in Dish Washer with up to 65° C degrees and industry dish washer up to 80° C for 3 minutes.

However, the cup becomes dull and the quality deteriorates, so it is recommended to wash it by hand.

HENROCUP® can handle temperature range between -20° C to 95° C.

## Warnings

Do not use any fluid in the HENROCUP® institution that is over 95° C in temperature.

**PLEASE DO NOTICE THAT SOME DRINKS MAY MISCOLOR THE INSIDE OF THE CUP.**

Dispose of the HENROCUP® institution if the surface does not feel smooth and has a milky-white color.

## Environmentally friendly disposal

Please dispose the cup in an environmentally friendly neutral way.

HENROCUP® Institution are made of recyclable material which should be recycled. Please therefore dispose the old HENROCUP® Institution via suitable collection systems.

**IF YOU HAVE ANY DOUBT OF WHEN TO DISPOSE THE HENROCUP® INSTITUTION, YOU CAN SEND AN EMAIL TO [HENROTEK@HENROTEK.COM](mailto:HENROTEK@HENROTEK.COM)**

<b>Technical datasheet HENROCUP® Institution</b>	
<b>Material Cup</b>	Copolyester
<b>Weight</b>	139,2 Gr
<b>Height</b>	9 cm
<b>Width</b>	15 cm
<b>BPA</b>	HENROCUP® Institution is BPA Free
<b>Marked</b>	<b>CE</b>
<b>Color</b>	Clear with ml
<b>Milliliters</b>	230 ml
<b>Dishwasher</b>	65° C degrees Industry Dish Washer up to 80° C for 3 minutes
<b>Microwave</b>	Yes
<b>Basic UDI-DI No.</b>	57048312BW
<b>UDI-DI HENROCUP institution</b>	05704831128845
<b>UDI-DI HENROCUP institution with ml</b>	05704831128852
<b>Standards and Regulations</b>	Regulation (EC) 1935/2004 of 27. October 2004 Regulation (EC) 2023/2006 of 22 December 2006 Danish executive order, BEK nr 681 of 25/05/2020 Regulation (EC) 10/2011 of 14 January 2011 Regulation (EU) 2017/745 of 5 April 2017

<b>Technical datasheet Lid</b>	
<b>Material Cup</b>	LLDPE (Polyethylen)
<b>Weight</b>	13 Gr
<b>Marked</b>	<b>CE</b>
<b>Dishwasher</b>	Dish washer 65° C 80° C degrees Industry Dish Washer up to 3 minutes
<b>Microwave</b>	No
<b>Basic UDI-DI No.</b>	57048311BU
<b>UDI-DI HENROCUP Lid small</b>	5704831000271
<b>UDI-DI HENROCUP Lid Big</b>	5704831000011
<b>Standards and Regulations</b>	Regulation (EC) 1935/2004 of 27. October 2004 Regulation (EC) 2023/2006 of 22 December 2006 Danish executive order, BEK nr 681 of 25/05/2020 Regulation (EC) 10/2011 of 14 January 2011 Regulation (EU) 2017/745 of 5 April 2017

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