
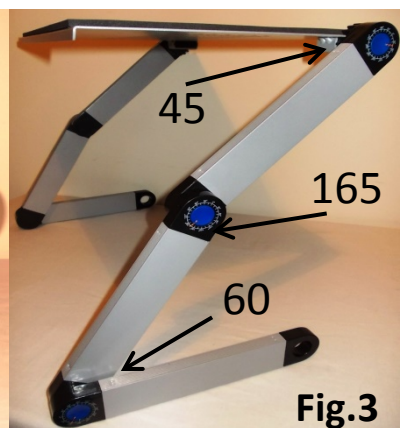


## Assembly Instructions – ARAT Table Shelf

Your ARAT Table-Shelf is used during performance of the ‘Grasp’ and ‘Pinch’ Test Sub-Scales. It should be stored fully folded, foam layer downwards in your case.

To erect your Table-Shelf correctly, please follow these instructions:

1. Place the foam table-top face downwards on a flat surface.
2. Gently depress the blue button (*Fig.1*) on the joint of the outermost, ‘lower’ leg (*Fig.2*) and rotate until all of the coloured dots align.
3. Release the blue button to ‘lock’ the leg at its angle of 60 degrees (*Fig.3*)
4. Repeat process on the centre ‘middle’ leg (*Fig. 2*) to an angle of 165 degrees (*Fig.3*)
5. Repeat process on the innermost ‘top’ leg (*Fig. 2*) to its angle of 45 degrees (*Fig.3*)
6. Repeat steps (2) to (5) to adjust and set the legs on the other side of the table.
7. To fold the Table-Shelf flat, return each joint to 0 degrees starting with the ‘top’ leg.

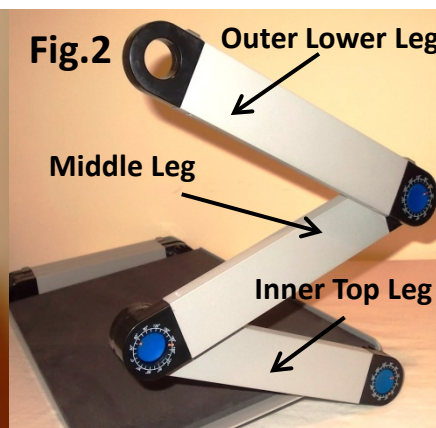

**Warning:** *This Table-Shelf must only be used in conjunction with an ‘Action Research Arm Test’ and must never be loaded with a weight of more than 5kg evenly distributed.*

**Neuroquip Ltd**

St John’s Innovation Centre, Cowley Road, Cambridge CB4 0WS, United Kingdom

Tel: (+44) 1223 420252 / (+44) 7763 003287

[www.neuroquip.co.uk](http://www.neuroquip.co.uk)