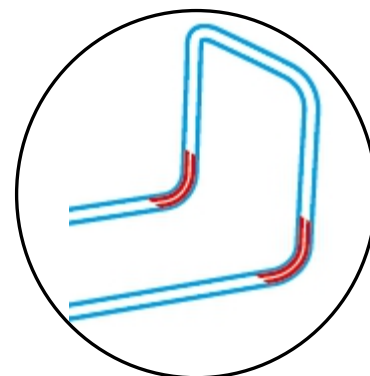
Bettgriff "FRIDA" Art.-Nr. BETTG010

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DE	Gebrauchsanweisung	(2-7)
EN	Instruction Manual	(8-12)
FR	Manuel d'instructions	(13-18)
IT	Istruzioni per l'uso	(19-24)
ES	Manual de uso	(25-29)
NL	Instructiehandleiding	(30-35)

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Instruction Manual
Bed handle "FRIDA"
Item number: BETTG010

1.0 Foreword

Dear customer,
we are pleased that you have chosen
a quality product from *rehastage GmbH*. A good relationship with our customers is of great
importance to *rehastage*. Customer proximity means: fast service, as little bureaucracy as possible
and close cooperation with our customers. Seek advice from your treating doctor or therapist to
ensure that this product is suitable for you and meets your needs.

**⚠ For your personal safety, please read these operating instructions carefully before using the
product. If you have any problems understanding the information, warnings and instructions
given here, please contact your specialist dealer to avoid possible injury or material damage due
to incorrect operation.**

Do not throw away these instructions for use, even after the aid has been installed, so that you can
use them later if necessary.

If you need spare parts or accessories, or simply have a question about your assistive device, we are
here for you.

We want you to be satisfied with our products and our service. Therefore, we at *rehastage* are
constantly working on the further development of our products. For this reason, changes in form,
technology and equipment may occur in our product range. Therefore, no claims can be derived
from the information and illustrations in these instructions for use.

2.0 Intended use

It is intended for use by nursing staff, family carers or by the user / patient themselves. It enables
the user / patient to get into a sitting position without assistance or as a support when getting in
and out of bed. The device is intended for use by an adult person / patient who does not suffer
from any contraindications mentioned.

- Indication:

The bed handle "FRIDA" (bed entry and exit aid) is an aid for users and / or patients with generally
reduced or impaired movements.

The provision of raising aids is indicated when, for example, due to a loss of balance, the position
between raising the upper body and achieving a safe sitting posture cannot be maintained. For
example, one arm must be able to bear sufficient weight and there must be sufficient trunk control
and stretching ability of the legs to enable the upper body to be raised largely independently.

- Contraindications:

⚠ The bed handle "FRIDA" is not suitable for people with Alzheimer's disease or dementia, who take
sedatives, are confused or frail. There is an increased risk of entrapment and injury here.

⚠ For people trying to climb over the product, there is an increased risk of injury from falling.

⚠ The device is NOT intended for use by children.

⚠ It is essential that you observe the further warnings and precautions listed under point 5!

The bed handle "FRIDA" is a class I medical device according to Regulation (EU) 2017/745 of the
European Parliament on medical devices.

3.0 Scope of delivery:

1 piece steel tube "handle part with foam rubber cover "

2 pieces steel tube "base piece long side "

1 piece steel tube U-shaped, "end piece "

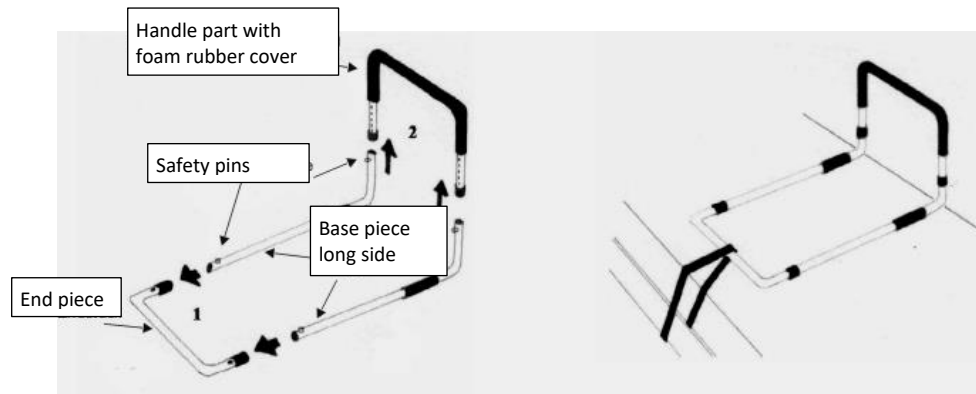
1 piece "belt", 140 cm

4.0 Assembly and installation instructions for the "FRIDA" bed handle

4.1 Assembling and mounting the bed handle

- 4.1.1 Place the *end piece* on a flat, large surface and insert one of the two *base pieces long side* into each end until you hear the safety pins click into place and they also **visibly protrude from the holes on the end piece**. Both *base pieces* are now securely connected to the *end piece*.
- 4.1.2 Next, insert the *handle part* **simultaneously** onto the two ends of the *base pieces*. Again, please check the correct engagement of the safety pins, which must visibly protrude from the *handle part* to ensure a secure hold.

4.2 Installation of the bed handle in the bed

- 4.2.1 Raise or lower the mattress and position the bed handle "FRIDA" on the inner bed frame. The *handle section* should be positioned so that the user can easily reach the handle in a lying position. **ATTENTION:** Keep a distance of at least 32 cm between the handle and the head or foot end of the bed to prevent trapping. The handle should protrude at least 10 cm above the surface of the mattress to prevent the user from climbing over it.
- 4.2.2 On the opposite side of the bed, place the *strap* around the *end piece* and attach it only to the stable and sound bed frame.
 - ⚠ **Do not install the aid without fastening the safety belt to the bed frame or slatted frame!**
- 4.2.3 Replace the mattress and make sure the bed handle is secure.
 - ⚠ **It is important that the bed handle is tight against the mattress!**Now the bed handle "FRIDA" can be used.

4.3 Changing the bed linen or the mattress

In general, it will not be necessary to remove the bed handle "FRIDA" for this purpose, as it only lies on the bed base and has no fixed connection with the mattress.

4.4 Instructions for dismantling the bed handle "FRIDA" (e.g. change of bed or taking it with you when travelling)

Lift the mattress or take it down. First loosen the strap on the end piece and the bed frame. Remove the complete bed handle and place it on a flat surface. Separate the handle section by pressing in the safety pins and slowly and smoothly pulling the handle section out of both base sections. Detach the base pieces from the end piece by first pressing in the safety pin on one side and slowly pulling out the base piece, then repeat this on the other side.

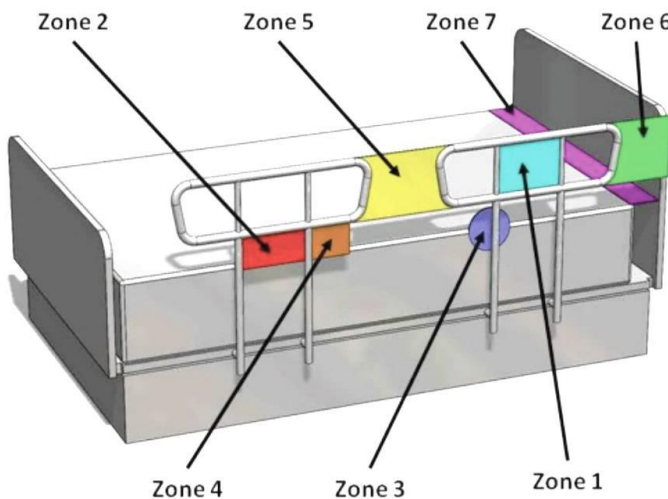
5.0 Precautions / Warnings

- ⚠ For safety reasons, it is important that the following conditions are met during installation and use in order to avoid the risk of injury or entrapment and any resulting risk of suffocation.

Instruction Manual
Bed handle "FRIDA"
Item number: BETTG010

5.1 Dimensions and distances to be observed (based on standard DIN EN 60601-2-52)

Zone 1: / Not applicable for this product.

Zone 2: / Not applicable for this product.

Zone 3: Between the bed handle and the mattress - This area between the inside of the bed handle and the mattress, if too large, can be a risk for head entrapment. The maximum allowable distance is 12 cm (4.75 in.).
However, if possible, the bed handle should be tight against the mattress.

Zone 4: / Not applicable for this product.

Zone 5: Between Split Bed Handles - When multiple bed handles are used on the same side of the bed, the space between the bed handles may pose a risk of head, neck, or chest entrapment. The distance between bed handles should be at least 32 cm (12.5 in.)

Zone 6: Between the end of the bed handle and the head of the bed - The recommended distance between the end of the bed handle and the head of the bed is at least 32 cm (12.5 in.), otherwise there is a risk of trapping the head, neck or chest.

Zone 7: Between the head or foot of the bed and the end of the mattress - The size of the mattress should fit the bed frame or base exactly. If the distance between the head or foot of the bed and the end of the mattress is too great, the risk of entrapment increases.

5.2 Mattress properties and requirements

There is an increased risk of entrapment and resulting injury when using mattresses that are too soft or too thin.

To ensure that the existing mattress is suitable for use with this product, test the firmness of your mattress by pressing a fist-sized object weighing approximately 5.4 kg (12 lbs) onto the edge of the mattress. If the sinking depth is more than 5 cm (2 inches), the mattress used is too soft and the bed rail should not be used.

The thickness of the mattress should be at least 20 cm (8 in.).

5.3 Warnings, notices and exclusions

- ⚠ The instructions accompanying the product must be followed carefully to ensure that the product is properly installed and assembled. Failure to follow the instructions may result in serious injury to the user(s) of the product.
- ⚠ The product is not intended to support the full weight of a person. This product is only intended as a balance aid and support when sitting down and getting up from bed.
- ⚠ The bed handle "FRIDA" must not be used on children's beds, bunk beds, water beds or inflatable beds.
- ⚠ Never attach the bed handle "FRIDA" to the bed, slatted frame or bed frame on the handle side. This can lead to serious injuries to the user(s) due to jamming.
- ⚠ Regularly check the positioning of the bed handle "FRIDA" in order to detect areas of possible entrapment at an early stage.
- ⚠ With electrically adjustable bed sections, make sure that you do not trap your limbs when raising or lowering the bed.

Instruction Manual
Bed handle "FRIDA"
Item number: BETTG010

- ⚠ Make sure that children do not play with or hang onto the product.
- ⚠ Additional safety measures should be taken for persons belonging to certain risk groups. These include persons with a mental impairment (including persons who are under the influence of medication), very tall persons or persons who are very overweight (which could change the gap between the bed handle "FRIDA" and the mattress), as well as persons who are generally very restless when awake or asleep.
- ⚠ If you have an illness or injury, or are taking medication that affects your sense of balance or your ability to stand or sit, please consult your physician first before using this product. Improper use of the product may result in serious injury or damage.
- **DISCLAIMER:** Neither Stander Inc. nor the manufacturer/distributor shall be liable for any damage or injury that may result or have resulted from the improper use or installation of the product.
- In the event of assembly problems, please contact the place of sale (specialist trade) or directly to:
Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de
- In the event of serious adverse events (deterioration of the user's condition) caused by the use of our product, we are obliged to report this to the Federal Institute for Drugs and Medical Devices. Therefore, we ask you to inform us directly in such a case:
Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de

6.0 Cleaning instructions

Clean the "FRIDA" bed handle regularly with a damp sponge, mild soap and a household disinfectant. Dry it with a dry soft cloth.

Do not use abrasive cleaning agents such as scouring milk or powder, this may destroy or damage the surfaces and threads.

7.0 Reuse / Lifetime

Due to its simple and solid construction, the bed handle "FRIDA" is suitable for reuse. The number of reuses depends on the condition of use of the device. Before each reuse, the bed handle "FRIDA" must be hygienically prepared and the technical and functional safety of the product checked and, if necessary, restored.

The assembly has to be carried out as described under point 4.

If the product is used as intended in accordance with the specifications, the lifetime is expected to be 5 years.

However, the effective service life may vary depending on the frequency and intensity of use as well as on the care and general condition.

8.0 Technical data:

Width:	51 cm
Length:	88 cm
Height of the handle:	adjustable 42 - 53 cm
Dead weight:	2.4 kg
Max. user weight:	up to 100 kg
Max. Load in use:	Depending on the mounting conditions

9.0 Disposal / recycling of materials

If the aid was provided to you free of charge, it may not belong to you. If it is no longer needed, follow the return instructions provided by the organization that provided you with the aid.

The following section describes the materials that were used for the resource, including

Instruction Manual
Bed handle "FRIDA"
Item number: BETTG010

Information on the disposal or recycling of the aid and its packaging. Special regulations may also apply locally with regard to disposal or recycling; these must be taken into account during disposal.

- -steel: bed handle "FRIDA"
- -plastic: belt, packing material
- -Carton: outer packaging

Disposal should be done through a disposal company or public disposal site.

10.0 Warranty

In addition to the statutory warranty, rehastage offers a warranty on this product for manufacturing or material defects of 2 years from the date of purchase. The warranty applies only to the first user of the item. Please keep the invoice, which is your proof of purchase. The warranty does not cover defects due to wear and tear, improper handling or commercial use of the item. The warranty is void if the item is repaired or constructively altered by anyone other than rehastage itself or a dealer authorized by rehastage. The warranty is void if repairs or modifications or replacement of individual parts that are not original rehastage spare parts are made by anyone other than the seller or supplying company. During the warranty period, rehastage shall decide whether to repair the device free of charge or replace it free of charge.

11.0 Explanation of symbols used, markings and information to be provided

Manufacturer, name and address of manufacturer	Manufacturer's authorised representative within the European Community	Medical device	Attention! Information such as warnings or precautions	Follow the instructions for use!

12.0 Accessories:

Glides for all bed handles and adjustable bed side rails:

Glides allow easy use of the height adjustable back of electrically adjustable beds while the base of the bed handle/side rail is under the mattress.

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