Handleiding
Instruction Manual

circulation^{maxx}TM reviver

NEUROMUSCULAR STIMULATOR

ANOTHER GREAT PRODUCT DISTRIBUTED BY:

bioenergiser
WOLFF & BÜCHER AG

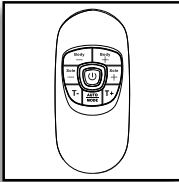
QUICK START GUIDE

PLEASE NOTE – THIS DEVICE DOES NOT VIBRATE – IT USES ELECTRICAL IMPULSES, NOT VIBRATION!

For detailed operation of your Circulation Reviver™ please refer to the comprehensive instructions within this manual.

To use Circulation Reviver™ immediately, follow this 5-point quick start guide.

1


Remove your Circulation Reviver™ from the packaging.

Take out the Remote Control and remove the screw from the back battery enclosure.


Insert 2 x AAA batteries into the compartment as per indication. Then screw up the battery enclosure.

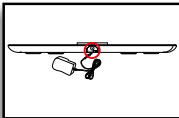2

Connect the DC adapter to a suitable mains outlet and plug the small DC socket into the device.

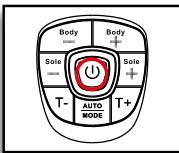3

Turn on the power the central display will light up orange.

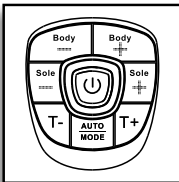
4

Remove your footwear and socks or stockings. Sit in a comfortable chair.

Both bare feet need to be on the device for it to work. Your left foot on the left foot plate, and your right foot on the right foot plate.

5

Increase the intensity levels for the foot by pressing the "SOLE+" or press "SOLE –" to decrease the intensity.


The intensity level ranges from 0-99, slowly increase the level until you begin to feel the micro-current stimulation.

CONTENTS

Important Safety information	3
Machine Overview and Parts Names	4
Functions of Control Panel	5
How to Operate	6 - 7
Troubleshooting	8
Technical Specifications	9

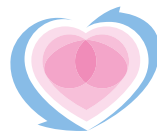SAFETY INFORMATION - PLEASE READ

Danger

This unit must not be used in combination with the following medical devices:

- Internally transplanted electronic medical devices, e.g. pacemakers
- Electronic life support equipment, such as respirators
- Electronic medical devices attached to the body, such as electrocardiographs

Using this unit with other electronic medical devices may cause erroneous operation of those devices.

Warning

Persons with the following conditions must consult the doctor before using this unit:

- Acute disease
- Malignant tumor
- Infectious disease
- Pregnancy
- Cardiac dysfunction
- High fever
- Abnormal blood pressure
- Skin sensory disorders or skin problems
- Receiving medical treatment, especially those feeling discomfort. May cause an accident or ill health.

Do not use this unit near the heart, above the neck, on the head, around the mouth or on diseased skin.

May cause an accident or ill health.

Application of electrodes near the thorax may increase the risk of cardiac fibrillation.

Do not use this unit simultaneously with other therapeutic devices or in combination with ointments including spray type ointments.

May cause discomfort or ill health.

Simultaneous connection of a PATIENT to h.f. surgical EQUIPMENT may result in burns at the site of the STIMULATOR electrodes and possible damage to the STIMULATOR.

Operation in close proximity (e.g. 1 m) to a shortwave or microwave therapy EQUIPMENT may produce instability the STIMULATOR output.

Do not use this unit for purposes other than treatment indicated in this manual. May lead to accident, problems, or failure of the unit.

Do not insert the electrode cord plug into any place other than the electrode cord jack of the main unit. May cause an electric shock or accident.

Do not disassemble or remodel this unit.

May cause fire, dysfunction, or accident.

Caution

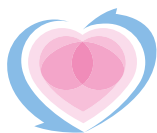
If the unit is not functioning properly or you feel discomfort, immediately stop using the unit.

If you feel any problems with your body or skin, consult the doctor and follow his/her instructions.







If you want to move the Electrode Pad to another region or your body during treatment, be sure to turn off the power. If not, you may receive a strong electrical shock.

Do not try to attach the Pads to any other person during the treatment. You may receive strong electrical shock.

Do not start treatment while wearing an electronic device. The settings and timings of the device may be affected.

MACHINE OVERVIEW AND PART NAMES

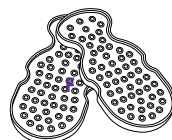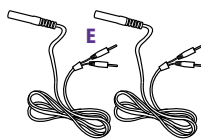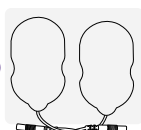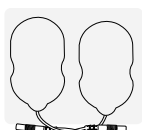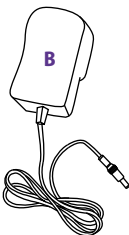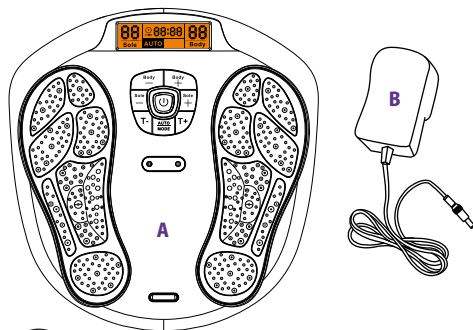A	Main Unit
B	AC Power Adapter
C	Remote Control
D	Electrode Gel Pad x 4 pcs.
E	Wire cable connecting the Electrode Gel pads and device
F	Plastic Protector for Gel Pads


TOP VIEW



1.	LCD Display Screen
2.	Control Panel
3.	Output for the feet Electrode Area for Left Foot
4.	Output for the feet Electrode Area for Right Foot
5.	Cable Connecting the Electrode Gel Pads and Device Jack A
6.	Cable Connecting the Electrode Gel Pads and Device Jack B
7.	Remote Control Receiver Sensor
8.	Silver colour decoration plate
9.	Adaptor Jack

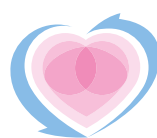SIDE VIEW


FUNCTION OF CONTROL PANEL

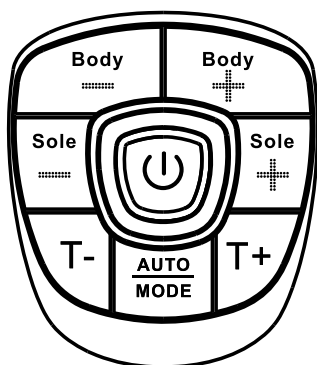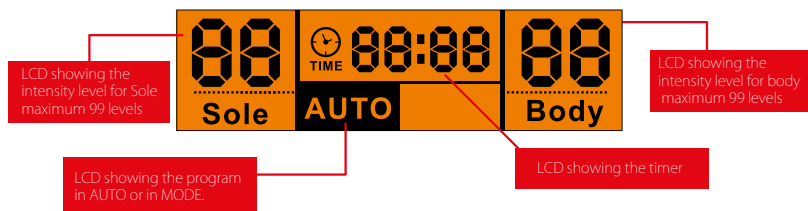
	ON/OFF switch button
BODY -	Decrease the output intensity of body (Available from 1 - 99 levels)
BODY +	Increase the output intensity of body (Available from 1 - 99 levels)
SOLE -	Decrease the output intensity of Sole (Available from 1 - 99 levels)
SOLE +	Increase the output intensity of sole (Available from 1 - 99 levels)
TIME -	Decrease the operation time (available from 1-60 minutes)
TIME +	Increase the operation time (available from 1-60 minutes)
AUTO/MODE	Auto - is the preset program with 14 pattern in cycle running for Foot and 10 pattern in cycle running for body Mode - user can fix the program to the exiting massage pattern on the rest of the time

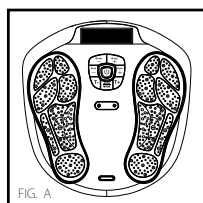Electrode area of the unit and the gel pad

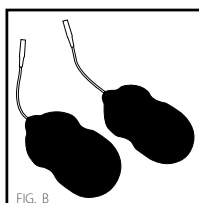On the device the black colour area on the device which is the electrode area for the sole. (see. the fig. A)

On the gel pad, the black colour area on the sticky part is the electrode area for the body, size is 5 cm x 9 cm. (see. the fig. B)

HOW TO OPERATE

For Foot – SOLE

Tips: In order to increase the conduction, apply **Circulation Reviver™ Foot gel on your sole.**

1. Place your bare feet onto Circulation Reviver™ (do not wear socks).
2. Press the on/off button, the LCD screen will light up in Orange. And the program show AUTO and both Band shown in 00, which is in standby mode. (see Fig. 1)
3. Gently increase intensity setting by pushing the button of "SOLE +". Or decrease intensity setting by pushing the button of "SOLE -". The intensity level is adjustable between 0 and 99. The LCD display will show the selected level. (see Fig. 2)
4. You can adjust the auto off timer by pressing the "Time -" or "Time +". Timer range from 1-60 minutes. To terminate the massage period, user can turn off the unit anytime by pressing the on/off Button once. And the timer start to count down from your time setting (see Fig.3)
5. If you satisfaction on the current massage program then you can lock the current massage programme by press the Auto/Mode key, then the rest of the massage time will only run the selected massage program. (see Fig. 4)

Important Information:

- A. The aim is not to get up to '99'.
- B. Chose an intensity level that is comfortable for you!
This level may vary from day to day.
- C. Remember to drink plenty of fluids – if you are dehydrated, this will reduce the effectiveness of the device.
- D. If you have dry feet – moisturise them or apply Circulation Reviver™ Foot Gel for maximum health benefits.

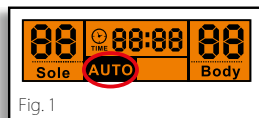
Fig. 1

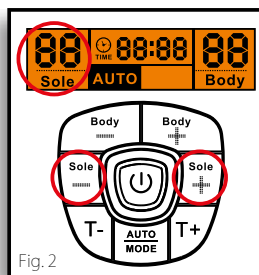
Fig. 2

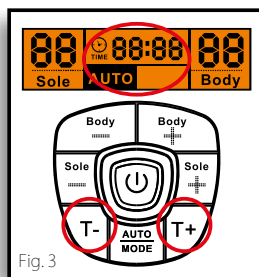
Fig. 3


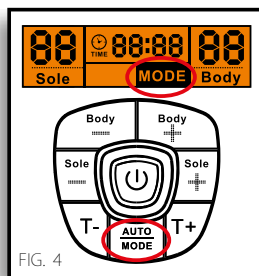
FIG. 4

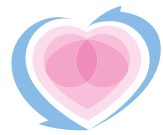FOR BODY

Using The Gel Pads

Wash and dry skin before use. Connect the output wire to the gel pads. Connect the other end of the output wire to the output jack on Circulation Reviver™. Remove the protective film from the adhesive pads. Attach the gel pads to the skin. Press the on/off button to turn on the unit and adjust the stimulating output intensity to the desired level. (The display will show the mode and level that you selected and start to count down).

1. Plug in the 2 cable into the cable jack on the unit. (see Fig. 5)
2. Connect pin of the cable to the gel pad properly. (see Fig.6)
3. Remove the protective film on the gel pad, and attached the 4 gel pads to the area of the body you wish to treat.
4. Repeat operation as in foot instructions, adjust the intensity using for body you wish to treat.
5. Gently increase intensity setting by pushing the button of "Body + ". Or decrease intensity setting by pushing the button of "Body - ". The LCD will also show the level which you have selected (see Fig. 7).
6. To terminate the massage period, user can turn off the unit anytime by pressing the on/off Button.

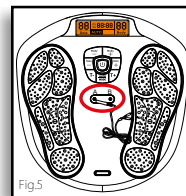
Fig.5

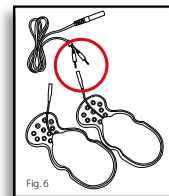
Fig. 6

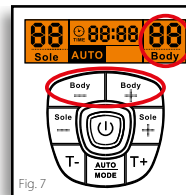
Fig. 7

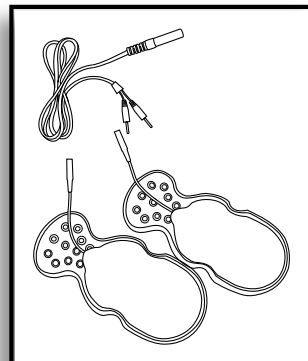
Care Of Your Gel Pads

Never stick two adhesive pads to each other. Keep the adhesive gel pads clean, never expose them to high temperature or direct sunshine. If the electrode gel pads are insufficiently adhesive or dirty, wipe with a wet cloth or change for new ones, replacement parts will be available directly from BioEnergisier or your distributor.

Do not clean the electrode gel pads with any chemical.

ALWAYS try and protect the gel pads. Store the gel pads on the gel pad protector when not in use, as in the illustration.

TROUBLESHOOTING

Troubleshooting & Maintenance

Problem	Cause	Rectification
Device will not turn on.	Batteries inserted in wrong direction. The adapter is not plug well into device properly.	Insert batteries in correct direction or check the battery is in full power. check the connection of the adapter jack is in well connected. And also the DC adapter with well connecting to the main socket.
Power turns off too soon	Gel pads not attached correctly to the skin.	Attach Gel pads correctly to the skin.
Power turns off while using massager	if you are using the batteries operation, then the batteries weak/exhausted.	Fit 4 new identical 1.5V alkaline batteries type AA.
	Treatment period of 30 minutes is over and power turns off automatically.	Restart treatment or turn off the massager.
	if you are using the body massage, the electrode gel pad may be broken	Replace electrode gel pad
It is difficult to attach Gel pad to the skin	Transparent film not peeled off	Peel off film on the adhesive surface of Gel pad
	Gel pad applied immediately after washing	Sufficiently dry Gel pad
	Adhesive surface of Gel pad damaged. The gel pads get dirty and lost their adhesive/stickiness.	Replace Gel pad Replace Gel pad or clean the gel pad with a small drop of water onto the sticky side of the electrode pad and rub into the surface
Adhesive surface of Gel pad is not sticky	Use of Gel pad during perspiring	Leave Gel pad in freezer for overnight
	Gel pad washed too long and/or too frequently	
	Gel pads stored under high temperature, high humidity, direct sunshine	
It is difficult to feel stimulation	You sole is too dry, not moisture enough Your sole is not placing on the foot pedal properly	put some water to your sole to moisture your sole. Ensure both of your sole are placed on each pedal properly. each
	Gel pads not attached correctly to the skin	Attach Gel pad firmly to the skin
	Gel pads overlap each other	Reattach Long Life pads with no overlap
	Electrode cord not connected correctly	Connect electrode cord correctly
	Applied intensity too weak	Increase the intensity by turning the dial switch
The skin turns red or the skin feels irritated	Adhesive surface of Gel pads dirty or dry	Wash adhesive surface of Gel pads softly with your fingertips for about 3 seconds under slow running water
	Adhesive surface of Gel pads damaged	Replace Gel pads

Hygiene


After using the product

Clean the device with a soft, damp cloth but make sure squeeze a cloth dry and to clean the foot pedal area. Store the Electrode Gel pads on the plastic pad protectors provided.

Storage

Keep the whole set of product clean and storage in a dust free and dry location under the condition


Storage temperature and humidity	-10° C to 60° C, 10% to 95% RH
Operating temperature and humidity	-5° C to 50° C, 30% to 90% RH

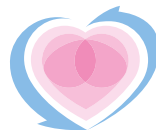TECHNICAL SPECIFICATIONS

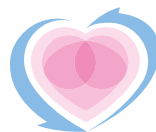Product Name	Circulation Reviver™	
Model	BE130	
Power Supply	6V DC or 4x1.5V batteries type AA* for the main unit 2x1.5V batteries type AAA* for the remote control	
Supplier of Adaptor	Golden Profit Electronics Ltd.	
Model no. of Adapter	GPE038-060050-3	
Adaptor Input	AC 100-240V~50-60Hz 0.1A	
Adapter Output	DC 6V 500mA 3.0W	
Battery life	>350 minutes	
Frequency Generation	Approx. 10 Hz to 55.56 Hz	
Power Consumption	1.05 W	
Maximum Output Voltage	U < 54.8V (during 1 kΩ load)	
Maximum Output Current	I < 910 μA (during 1 kΩ load)	
Operating Temperature And Humidity	-5° C to 50° C, 30% to 90% RH	
Storage Temperature And Humidity	-10° C to 60° C, 10% to 95% RH	
Main Unit Dimensions	338(L) x 324 (W) x 48(H) mm	
Weight Approx.	950 g	
Package Contents	Quantity	Parts
	1	Circulation Reviver™
	1	AC/DC Adaptor
	1	Remote Control
	2	Cable Wire for Electrode Gel Pads
	4	Electrode Gel Pads
	2	Plastic Gel pads protector
	1	Instruction Manual
	Accessories : Only use original accessories. If user use of other parts or materials can degrade minimum safety. Check that the contents of the delivery are complete.	

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© Bioenergiser™ Circulation Reviver™ are trademarks of
Bioenergiser (UK) Ltd. All rights reserved.
Bioenergiser Ltd.
Manufactured for Bioenergiser (UK) Ltd. in the PPR