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Assembly instruction
Additional handle "FRIDOLIN" (accessories)
Article: FRIDOLIN 010

Attention! This product can only be used in conjunction with the bed handle "FRIDA" (Rehastage Article: FRIDOLIN 010).

1.0 Foreword Dear customer, we are pleased that you have chosen a quality product from *rehastage GmbH*. A good relationship with our customers is of great importance for *rehastage*. Customer proximity means: fast service, as little bureaucracy as possible and close cooperation with our customers. Seek advice from your attending physician or therapist to make sure this product is right for you and meets your needs.

⚠ For your personal safety, please read these instructions for use carefully before using the product. If there are comprehension problems with the information, warnings and instructions listed here, please inquire with your specialist dealer to avoid any injuries or property damage caused by incorrect operation.

Do not throw away these instructions for use even after assembling the aid so that you can use them later if necessary.

If you need spare parts or accessories or simply have a question about your aid, we are there for you. We want you to be satisfied with our products and service. We at *rehastage* are therefore constantly working on the further development of our products. For this reason, changes in form, technology and equipment may occur in our product range. Therefore, no claims can be derived from the information and illustrations in these instructions for use.

2.0 Intended use

The use is carried out by the user / patient himself.

The product enables the user / patient in conjunction with the bed handle "FRIDA" to bring me into a sitting position without outside help or as a support when getting in and out of or into the bed as well as for positioning in bed. The additional handle "FRIDOLIN" in conjunction with the bed handle "FRIDA" is intended for use by an adult person / patient who does not suffer from the mentioned contraindications.

The additional handle "FRIDOLIN" is available in conjunction with the bed handle "FRIDA" for bed or bed grips. Mattresses wide of 90 cm or 100 cm suitable.

- **Indication:**

The additional handle "FRIDOLIN" in conjunction with the bed handle "FRIDA" (bed entry and exit aid) is an aid for users and / or patients with generally reduced or impaired movements.

The supply of erecting aids is indicated if, for example, due to a loss of balance, the position between raising the upper body and achieving a secure sitting posture cannot be maintained. Thus, an arm must be able to be loaded sufficiently and a sufficient trunk control and extensibility of the legs must be given to enable a largely independent straightening of the upper body.

- **Contraindications:**

⚠ The additional handle "FRIDOLIN" in combination with the bed handle "FRIDA" is not suitable for people with Alzheimer's or dementia who take sedatives, are confused or frail. There is an increased risk of entrapment and injury here.

⚠ For people who try to climb over the product, there is an increased risk of injury from traps.

⚠ The product is NOT intended for use by children.

⚠ Be sure to follow the warnings and precautions listed!


The additional handle "FRIDOLIN" is a Class I medical device according to Regulation (EU) 2017/745 of the European Parliament on medical devices

Assembly instruction
Additional handle "FRIDOLIN" (accessories)
Article: FRIDOLIN 010

3.0 Scope of delivery:


Part A	Handle bracket with cover	1 piece
Part B	Connectors	2 pieces
Part C	Security strap 60 cm long	1 piece

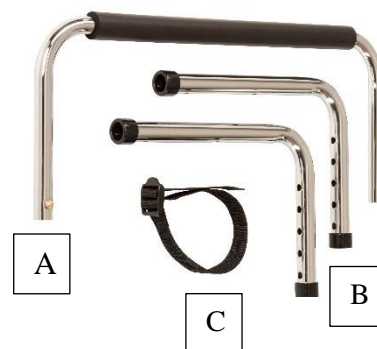4.0 Assembly and assembly instructions for the additional handle "FRIDOLIN" in conjunction with the bed handle "FRIDA". Initial situation, the bed handle FRIDA is fully assembled.

4.1 Assembly of the additional handle "FRIDOLIN" with the bed handle "FRIDA"

4.1.1 Plug in the **Connectors (B)** of the additional handle on the ends of the **Handle bracket with coating (A)**. Make sure that you hear the locking of the safety pins and that they also **protrude visibly from the holes at the end piece!**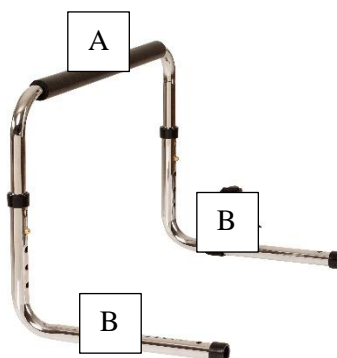


4.1.2 Now take the bed handle "FRIDA" and remove the end piece. If the bed handle "FRIDA" is already mounted in bed, please remove the safety belt beforehand.

4.1.3 Now remove the end piece (small pipe bend) from the bed handle "FRIDA" by pressing in the safety pins and pulling the end piece down from the base parts.

4.1.4 Place the bed handle "FRIDA" on a flat surface. Insert the two ends of the **connectors (B)** of the additional handle "FRIDOLIN" onto the base pieces of the bed handle "FRIDA". Snap the safety pins to the desired width (bed / mattress width 90 cm or 100 cm).

⚠ Make sure that the safety pins also protrude visibly from the respective holes of the connectors!

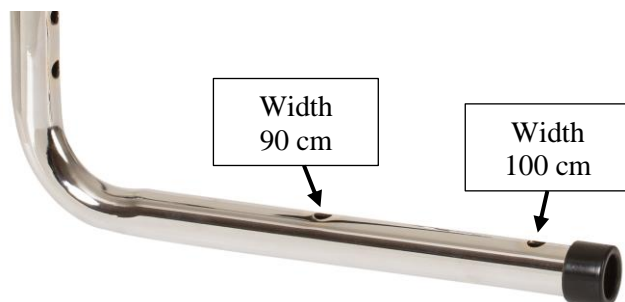Assembly instruction
Additional handle "FRIDOLIN" (accessories)
Article: FRIDOLIN 010

4.2 Installation of the bed handle in the bed

4.2.1 Lift the mattress or take it down and position the bed handle "FRIDA" with the mounted additional handle "FRIDOLIN" on the slatted frame of the bed.

We recommend positioning the handle parts in such a way that the user can easily reach the handle in a lying position and the handle part of the bed handle "FRIDA" is located on the headboard side of the bed when sitting on the edge of the bed.


- ⚠ **Note that the bed handle "FRIDA" is located on the entry and exit side of the bed!**
- ⚠ **ATTENTION! The distance between the upper edge of the mattress and the handle bar of the Fridolin must not exceed 12 cm.**
- ⚠ **Under all circumstances, comply with the dimensions and distances described in section 5.0 and observe the warnings, precautions to avoid risks or dangers that may lead to physical damage or deterioration of the user's state of health.**

4.2.2 Now attach the additional handle "FRIDOLIN" to the head side of the bed with the enclosed **safety belt (C)** to the slatted frame.

- ⚠ **Please note that with slatted frames with adjustable headboard, the attachment may only be made to the frame of the headboard, otherwise they block the adjustment function. In the case of electrically adjustable slatted frames, incorrect fastening can lead to a defect in the drive. Rehastage assumes no liability for damages resulting from faulty assembly.**
- ⚠ **Do not use the aid until you have attached it to the bed frame or slatted frame with the safety belt!**

4.2.3 Put the mattress back on and make sure the bed handle is secure.

- ⚠ **It is important that the bed handles tightly attached to the mattress!**

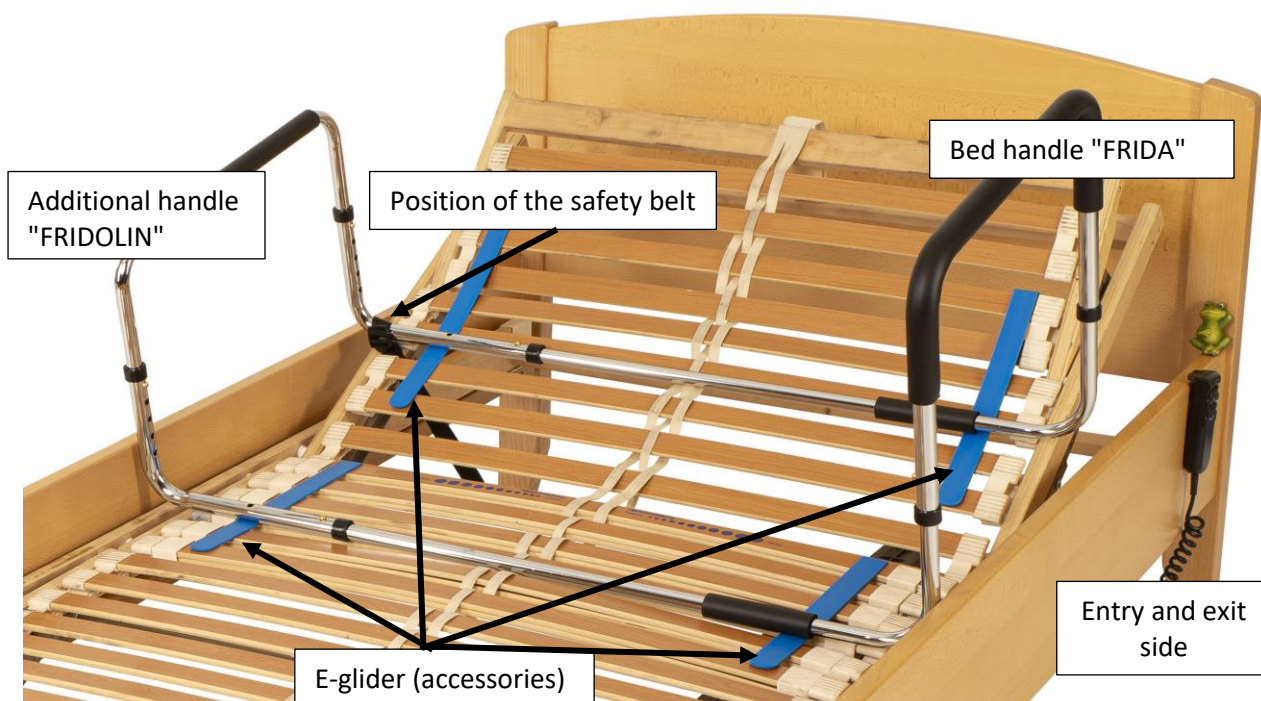We recommend the use of our "E-GLEITER", especially when using adjustable slatted frames. These are available as accessories from specialist retailers (see pt. 12).

4.3 Change of bedlinen or mattress

As a rule, it will not be necessary to remove the bed handle "FRIDA" with the additional handle "FRIDOLIN", as it is only on the bed base and has no firm connection to the mattress.

4.4 Dismantling of the bed handle e (e. g. bed change or transport when travelling)


Assembly instruction
Additional handle "FRIDOLIN" (accessories)
Article: FRIDOLIN 010

Lift the mattress or take it down. First, loosen the strap on the end piece and the bed frame. Remove the entire bed handle and place it on a flat surface. The components can be separated by pressing in the safety pins and slowly and evenly pulling the individual components apart.

5.0 Precautions / Warnings

⚠ **For safety reasons, it is important that the following conditions are met during assembly and during use in order to avoid the risk of injury or entrapment and, if necessary, the resulting risk of suffocation.**

5.1 Dimensions and distances to be observed (based on standard DIN EN 60601-2-52)

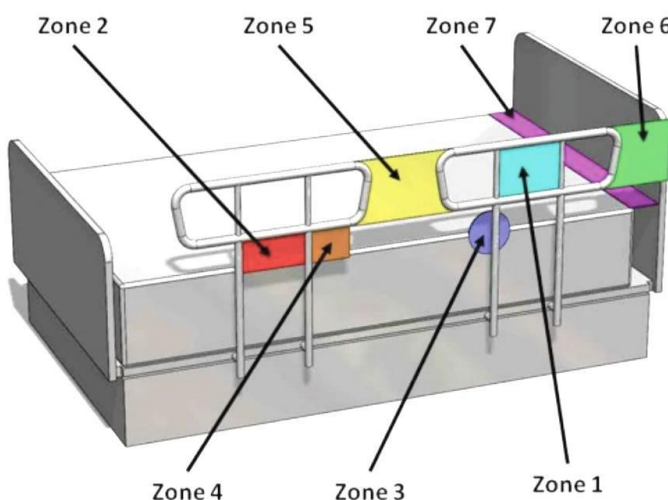Zone 1: / Not applicable to this product.

Zone 2: / Not applicable to this product.

Zone 3: Between the bed handle and the mattress – This area between the inside of the bed handle and the mattress, if too large, can be a risk of pinching the head. The maximum permissible distance is 12 cm (4.75 in.).
If possible, however, the bed handle should fit tightly on the mattress.

Zone 4: / Not applicable to this product.

Zone 5: Between split bed handles – If multiple bed handles are used on the same side of the bed, the space between the bed handles can pose a risk of pinching the head, neck, or chest. The distance between the bed handles should be at least 32 cm (12.5 in.)

Zone 6: Between the end of the bed handle and the headboard of the bed – The recommended distance between the end of the bed handle and the headboard of the bed is min. 32 cm (12.5 in.), otherwise there is a risk of pinching of the head, neck or chest.

Zone 7: Between the head or foot part of the bed and the end of the mattress – The size of the mattress should be precisely matched to the bed frame, substructure. If the distance between the head or foot part of the bed and the end of the mattress is too large, the risk of pinching increases.

5.2 Mattress characteristics and requirements

There is an increased risk of pinching and resulting injuries when using mattresses that are too soft or too thin.

To ensure that the existing mattress is suitable for use with this product, test the strength of your mattress by pressing a fist-sized item weighing approximately 5.4 kg (12 lbs) onto the edge of the mattress. If the sinking depth is more than 5 cm (2 inches), the mattress used is too soft and it should be refrained from using the bed rail.

The thickness of the mattress should be at least 20 cm (8 in.).

5.3 Warnings, Notices, and Exclusions

⚠ The instructions associated with the product must be followed carefully to ensure that the product is properly installed and assembled. Failure to follow the instructions may result in serious injury to the user(s) of the product.

⚠ The product is not intended to carry the full weight of a person. This product only serves as a balance aid and support when sitting down and getting up from the bed.

Assembly instruction

Additional handle "FRIDOLIN" (accessories)

Article: FRIDOLIN 010

- ⚠ The bed handle "FRIDA" in conjunction with the additional handle "FRIDOLIN" may not be used on cots, bunk beds, waterbeds or inflatable beds.
- ⚠ Under no circumstances attach the product additionally to the entry and exit side of the bed, slatted frame or bed frame. This can lead to serious injuries to the user, e. g. due to pinching.
- ⚠ Check the positioning of the bed handle regularly to detect areas where pinching is possible at an early stage.
- ⚠ With electrically adjustable bed parts, make sure that you do not pinch your limbs when you start up and down.
- ⚠ Make sure that children do not play with the product or get attached to it.
- ⚠ Additional safety measures should be taken for persons belonging to certain risk groups. These include people with a mental impairment (including those under the influence of medication), very tall people or people with severe obesity (which could change the gap between the bed handle and the mattress), and people who are generally very restless when awake or sleeping.
- ⚠ If you have an illness or injury, or are taking medications that affect your sense of balance or your ability to stand or sit, please consult your physician before using this product. Improper use of the product can lead to serious injury or damage.
- **DISCLAIMER:** Neither the manufacturer nor the distributor is liable for damages or injuries that may or may arise from the improper use or assembly of the product.
- In case of installation problems, please contact the sales office (specialist trade) or directly to:
Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de
- In the event of serious undesirable incidents (deterioration of the user's condition) caused by the use of our product, we are obliged to report this to the Federal Institute for Drugs and Medical Devices. Therefore, we ask you to inform us directly in such a case:
Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de

6.0 Cleaning instructions

Clean the bed handle regularly with a moist sponge, mild soap and a household disinfectant. Dry it with a dry soft cloth. Do not use abrasive detergents such as scouring milk or powder, this can lead to destruction or damage to the surfaces and threads.

7.0 Reuse / Lifetime

Due to its simple and solid construction, the additional handle "FRIDOLIN" is suitable for reuse. The number of re-inserts depends on the state of use of the device. Before each reuse, the additional handle "FRIDOLIN" must be hygienically prepared and the technical and functional safety of the product must be checked and, if necessary, restored.

The assembly must be carried out as described in point 4.

If the product is used as intended in accordance with the specifications, the service life is expected to be 5 years

However, the effective service life can vary depending on the frequency and intensity of use as well as the care and general condition.

8.0 Technical data:

Width:	51 cm
Length:	28 cm
Height of the handle:	adjustable 26 – 36.5 cm
Net weight:	1.0 kg

Assembly instruction
Additional handle "FRIDOLIN" (accessories)
Article: FRIDOLIN 010

User weight: up to 100 kg
 Load in use: Depending on the installation conditions

9.0 Disposal / recycling of materials

If the aid has been provided to you free of charge, it may not be yours. If it is no longer needed, please follow the instructions you received from the organization that provided you with the tool regarding the return. The following section describes the materials used for the tool, using information on the disposal or recycling of the aid and its packaging. Special regulations regarding disposal or recycling may also apply on site, which must be taken into account during disposal.

- Steel: Additional handle "FRIDOLIN"
- Plastic: cover handle, strap, packaging material
- Carton: outer packaging

Disposal should be carried out by a disposal company or a public disposal point.

10.0 Warranty

In addition to the statutory warranty, rehastage offers a warranty on this product against manufacturing or material defects of 2 years from the date of purchase. The warranty only applies to the first user of the item. Keep the invoice well, which is your proof of purchase. The warranty does not cover defects caused by wear and tear, improper handling or commercial use of the device. The warranty expires if the item is repaired or structurally altered by anyone other than rehastage itself or a dealer authorized by rehastage. The warranty obligation expires if repairs or changes or replacement of individual parts that are not original rehastage spare parts are carried out by parties other than the seller or the supplying company. During the warranty period, rehastage is responsible for deciding to repair the device free of charge or to replace it free of charge.

11.0 Explanation of symbols, markings and information to be provided

				
Manufacturer, name and address of the manufacturer	Unique Device Identification Product Identification of Medical Devices	Medical	Attention! Information on warnings and precautions	Follow the instructions for use!

12.0 Accessories:

Glides for all bed handles and adjustable bed side rails.
 The glides allow easy use of the height-adjustable back section of electrically adjustable beds, while the base of the bed handle/side rail lies under the mattress.
 Art. No. = E-Gleiter

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