


Evacuation Sheet

Flame retardant
Low friction
Rescue situation



The Evacuation Sheet is used in facilities with clients who are always bedridden. In a rescue situation the Evacuation Sheet can very easily and safely be dragged over floors, tiles, carpeting, down stairs and out of the hazardous area by a single person. By holding large belt grips at the top or bottom ends and at the centre of the two sides the mattress is pulled out of the bed across the floor into a safe zone. Approved according to DIN1865, Patient Handling equipment for rescue services/ambulances.

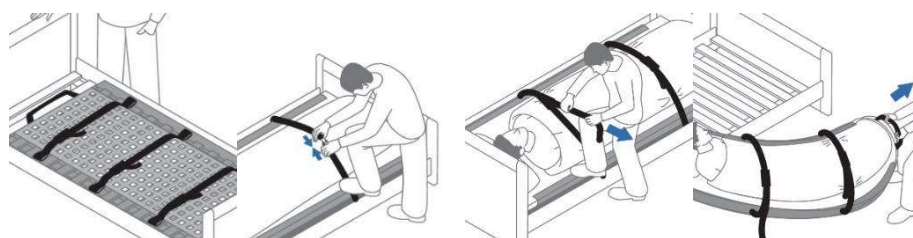

Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Evacuation Sheet	PVC-coated polyester weave, with ventilation openings, latex-free	white	200 kg	10 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-EVA-7053-H	200 x 87 For bed size: L: 190 – 205 W: 80 – 100	1710 g	(01)04250355370532(10)LOT EVACMS
PM-EVA-7053-S	200 x 100 For bed size: L: 190 – 205 W: 100 – 140	1920 g	(01)04250355370549(10)LOT EVACM

Hygiene and maintenance

The Evacuation sheet can be wet cleaned. Use warm water and a mild cleaning agent. If applicable, wipe the product off with a moist cloth and then let it dry thoroughly. Don't use any scouring or bleaching agents or any sharp-edged objects for cleaning. All Evacuation Sheets can be machine washed at 60°C using a mesh laundry bag. The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the product properties. When necessary, we recommend wipe disinfection with the product Trionic® or comparable products.



made in Germany



(wash in laundry net)