Urias[®] Inflatable Splints

Pocket Pressure Gauge Instructions for Use


Inflate the 'Urias[®]' Air Splint in the usual manner as detailed in the instructions for use, i.e. by opening the 'T' valve, inflating by mouth to the desired or recommended pressure, and then closing the 'T' valve.

Ensure that the Urias[®] Splint is inflated to the recommended / desired level by using the Pocket Pressure Gauge.

- 1 Push the connector of the gauge over the Urias[®] splint inlet tube and open the 'T' valve.
- 2 Read the pressure on the gauge dial and close the 'T' valve.
- 3 If more pressure is required, remove the gauge from the Urias[®] splint inlet tube and inflate the splint further by mouth and repeat steps 1 and 2 above.
- 4 If less pressure is required, remove the pressure gauge from the Urias[®] splint inlet tube and open the 'T' valve of the splint to release air. Repeat steps 1 and 2 above.
- 5 Repeat process until the required pressure level is achieved.

The Pocket Pressure Gauge is designed to measure the pressure in the 'Urias[®]' splints to the recommended 40mm Hg following *inflation by mouth*.

Care should be taken not to 'kink' the connecting tube of the Pressure Gauge especially when not in use.

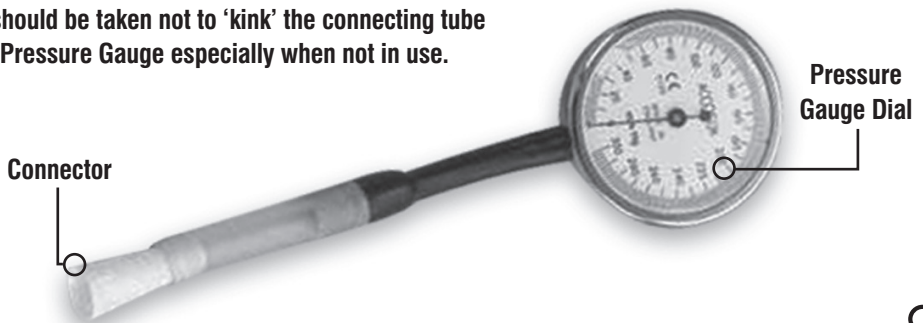Arden Medical

CE

Supplied By:

Arden Medical Ltd, Arden Road, Arden Forest Industrial Estate,
Alcester, Warwickshire B49 6HN, England

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