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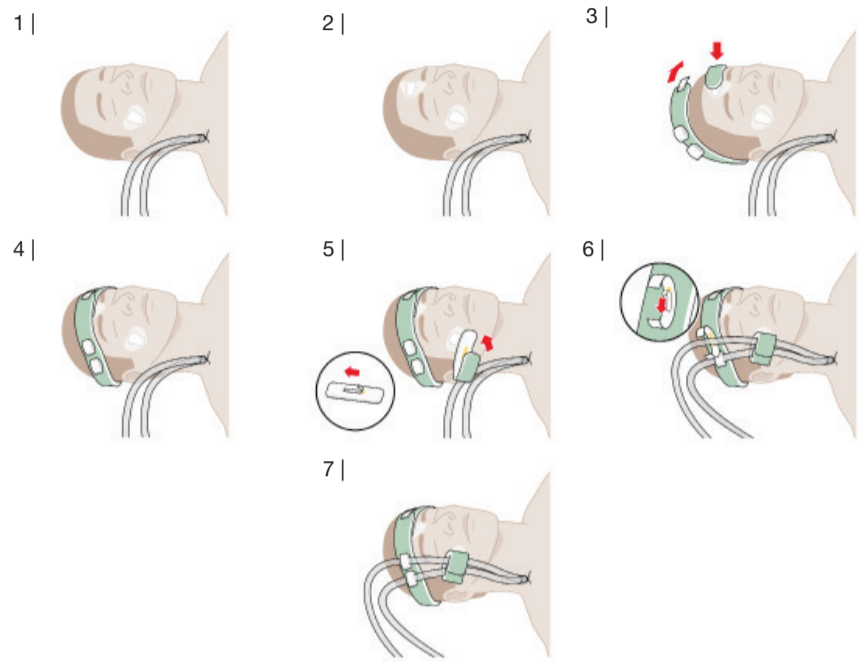
1. Product Description

ECMO-Fix is used for fixing and securing catheter tubing sets, which are applied during ECMO / ECLS therapy for alert or sedated patients. ECMO-Fix minimizes the tensile and lifting forces at the puncture. The fixation can be optimally adapted to the head and leg circumferences and is suitable for both adults and adolescents. The adjustable hook and loop straps enable an individual adaption to various charrière sizes.

2. Application Areas

ECMO-Fix 1:

Fixing and securing catheter tubing sets in the head area when the cannulation is made via the jugular vein.

ECMO-Fix 2:

Fixing and securing catheter tubing sets on the leg when the cannulation is made via the femoral vein. ECMO-Fix 2 can be also applied for the fixation of pumpless systems (pECLA).

ECMO-Fix 1 and ECMO-Fix 2 can be used for both single lumen and dual lumen catheters and tubing systems.

3. Product Versions

The different sets consist of following components:

ECMO-Fix 1:

- One reclosable fixation band with two sewed hook and loop straps with integrated adherend
- Two TuBo-Lock plasters with hook and loop surface
- One reclosable fixation strap with integrated adherend

ECMO-Fix 2:

- One reclosable fixation band with two sewed hook and loop straps with integrated adherend
- One Main-Lock Set 14 adhesive tape

Composition of Main-Lock Set 14

The set consists of two components:

- Adhesive tape with a velour fleece surface
- Hook and loop connection

Main-Lock Set 14 serves for the fixation of catheters, drainages and tubing systems with different charrière sizes.

4. Warning Notices

ECMO-Fix is NO alternative to a necessary suture for fixing the ECMO cannula but can additionally support a suture. All components of the ECMO-Fix set are non-sterile and must NOT have direct contact to the puncture. TuBo-Lock and Main-Lock plasters may only be used on unaffected skin. Remove the fixation set with allergic reactions and/or cutaneous reactions.

5. Shelf Life

The expiry date is imprinted on the back of the plaster packaging. If the expiry date is exceeded, the fixing ability of the plasters can diminish. The shelf life of the foam components is not limited when complying with the storage recommendation.

6. Traceability

Each TuBo-Lock and Main-Lock plaster has a LOT number on its primary packaging.

7. Handling

Preparation

Before applying TuBo-Lock respectively Main-Lock plasters prepare the patient's skin according to your hospital standards. The skin has to be clean, dry and free of grease. Residues of disinfectant, plaster removers or benzine respectively surgical spirit have a negative impact on the adhesion and are to be removed completely from the skin before applying TuBo-Lock respectively Main-Lock plasters. In order to guarantee an optimal adhesion, a hair removal is eventually necessary.

Application

ECMO-Fix 1

- 1 | Stick a TuBo-Lock plaster onto the patient's cheek on the ECMO cannula side.

- 2 | The second TuBo-Lock plaster is placed on the forehead.

- 3 | Fix the end without hook and loop fastener on the hook and loop surface of the plaster on the forehead and put the fixation band around the head.

- 4 | Make sure that the hook and loop straps are on the same level as the puncture of the ECMO cannula. Close the fixation band with its hook and loop fastener.

- 5 | Fix the fixation strap onto the hook and loop surface of the cheek plaster and remove the protective paper from the adherend within the fixation strap. Put the ECMO cannula into the fixation strap and close it.

- 6 | Remove the protective paper from the adherend of one of the hook and loop straps of the fixation band. Put the ECMO cannula into the hook and loop strap and close it. Repeat the procedure with the second hook and loop strap when using dual lumen ECMO cannulae.

- 7 | Fixation in final position.

ECMO-Fix 2

- 1 | Place the fixation band above the knee around the thigh. Close the fixation band with the hook and loop fastener.

- 2 | Open one of the hook and loop straps and remove the protective paper from the adherend.

- 3 | Put the ECMO cannula into the hook and loop strap and close it.

- 4 | Stick the adhesive tape of Main-Lock Set 14 centrally onto the thigh above the fixation band.

- 5 | Remove the pink protective film from the gel layer of the hook and loop connection and place the ECMO cannula onto the gel layer.

- 6 | Pull the longer end of the hook and loop connection through the eyelet. Fix the hook and loop connection with its hook side on the fleece surface of the adhesive tape.

- 7 | Fixation in final position.

Removal

The fixation bands and plasters can be removed from the skin without trauma and without adjuvants. A special plaster remover or benzine respectively surgical spirit is not necessary and not recommended.

Replacement Frequency

In order to maintain the optimal fixing ability, the fixation bands should be replaced at least once every 24 hours, TuBo-Lock and Main-Lock plasters at least once every 72 hours.

8. Legal Notices

The manufacturer Novo Klinik-Service GmbH accepts no liability for damages (in particular for functional failures, injuries, infections and/or other complications or other undesirable incidents) caused by unauthorized product modifications or improper handling, care (cleansing/disinfection) and/or storage of the products against the provisions of this instruction for use. Where legally permissible this is valid for resulting damages to the products themselves as well as for all resulting consequential damages. Product modifications on the part of the manufacturer are reserved at any time.

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